

# COVID19 **FACESHIELD** BPC001

## Instructions for Use

This FaceShield is supplied as a single-use product and should be used according to your local guidelines.

It is intended for general use but does not guarantee that particulates may reach areas of the face, eyes or respiratory system. It can be used in conjunction with face masks, goggles, safety glasses or corrective eyewear.

## Fitting

Remove the FaceShield from the packaging, apply one of the identification stickers to the top of the FaceShield and write your name in the space allocated.

Place the soft foam of the FaceShield directly against your forehead in a position that is comfortable and will not inhibit the wearer from being able to perform their normal duties.

Wrap the elasticated strap around the back of the head so as the FaceShield is held firmly but comfortably in place. Your FaceShield is now ready for use.

## Care and Storage of your FaceShield

This FaceShield is designed and manufactured to be single use/disposable and as such the following does not apply.

Try to avoid touching the centre of the FaceShield to avoid marking the visor.

The FaceShield should be stored to avoid unnecessary scratches or marks carefully, however slight scratching will not affect its use.

The FaceShield screen may be washed with soapy water or any alcohol-based cleaning product however do not use abrasive or scouring materials to clean it.

The FaceShield should be replaced if damaged or clarity of vision is hindered due to damage, or the strap has been damaged.

## Note

This FaceShield does not protect the wearer from laser radiation and appropriate eye protection must be worn.

## Disposal of your FaceShield

Disposal of your FaceShield should be done in accordance with local regulations and guidelines for medical waste.

Product Materials:

Shield. APET

Headband. PU Foam



Strap: Elastic

# COVID19 FACESHIELD <sup>☞</sup> BPC001

## Technical Specification

- PPE Category 1
- OSHA level splash protection
- Prevents exposure to blood borne pathogens and other infections
- Open sides provide free circulation
- Elastic band and foam forehead pad for comfort
- Designed for procedures and environments where risk of exposure to blood and / or bodily fluids through splashing is possible
- Equipped with a foam band and headband for comfort
- Can be worn with or without glasses or goggles
- Resistant to fogging
- Latex free
- CE marked
- Manufactured to ISO certified standards
- 20 face shields per case

### BPC001 300mu APET PPE FaceShield.

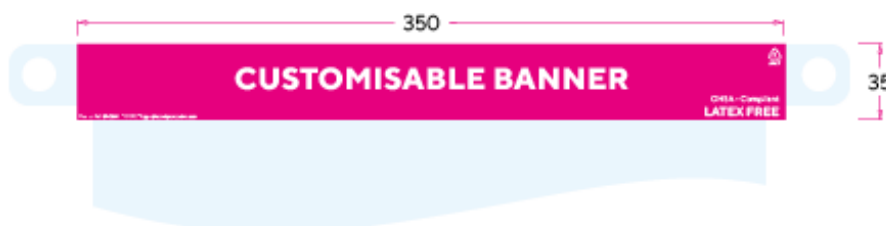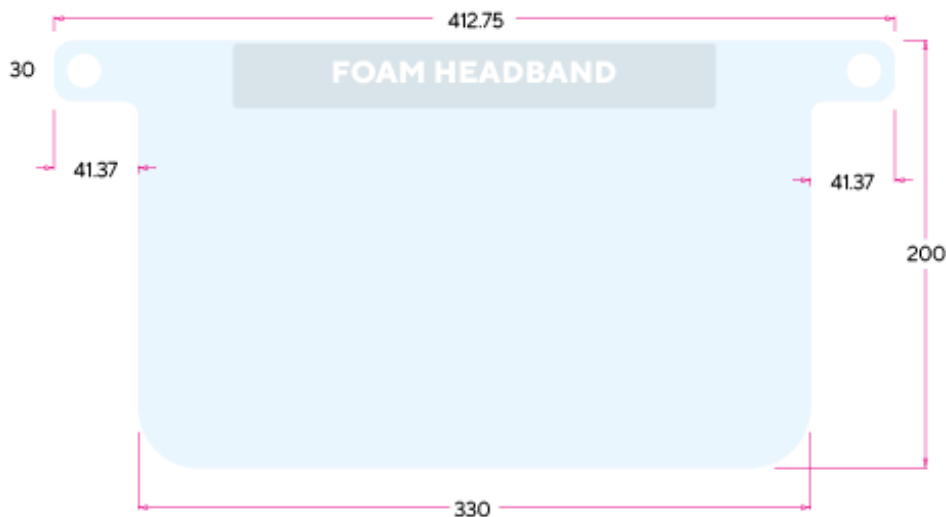

All dimension in mm. Drawing not to scale.



# EU Declaration of Conformity

Manufacturer:



Product Description:

Single use face shield

Product Name:



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
COVID19**FACESHIELD** ® BPC001

Classification:

Class 1 Medical Device

It is hereby declared that the medical devices specified above conforms with the essential requirements listed in Annex 1 of EC Directive 93/42/EEC of 14 June 1993 as amended by 2007/47/EC.

Signed:



Issue date:

14 April 2020

# REDACTED COPY

Unredacted EU Declaration of Conformity available upon request.

# COVID19 **FACESHIELD** BPC001

## Description

The visor of the face shield is manufactured from 250-300micron Amorphous PET (APET) which provides OSHA-level splash protection preventing exposure to blood-borne pathogens and other infectious diseases. The open sides of the Face Shields create air flow to prevent the mask from fogging. The wearer is protected from airborne particulate which may be exhaled from a patient with no loss of vision to the user.

The data sheet for the APET is available upon request.

## Comfort Band

There is a foam forehead pad which together with an elastic band to secure the Disposable Face Shield to provide comfort during use.

The foam forehead pad is comprised of strips of SR75, a light Grey 24kg/m<sup>3</sup> foam, with dimensions 245mm x 35mm x 35mm per strip.

The data sheet for the foam is applicable upon request.

## Elastic Shield Closing band

An elasticated comfort band is attached to the shield by the use of Velcro which removes the possibility of any metal staple either hurting the user or falling out and affecting the patient, the shield arrives pre-assembled to fit most users comfortably but if required the user can easily adjust the band and reposition to suit.

The data sheet for the elastic comfort band is available upon request.

## Packaging

20 Face Shields are packed in each box which is lined for protection of the face shield during storage and transport. A product identification label is on the outside of each box showing the following information:

- Manufacturer name and address
- Product code
- Lot number
- Date of manufacture

## Order

For distribution and order enquiries in England, Scotland and Wales please contact [ppe@brandprintcolour.com](mailto:ppe@brandprintcolour.com)